Canine Parvovirus Antigen Test Kit

For veterinary use only.

English version 

SNAP® Parvo

The SNAP Canine Parvovirus Antigen Test Kit is a rapid enzyme immunoassay for the detection of canine parvovirus (CPV) antigen in canine feces. This test detects a specific protein of CPV (including intact virus particles) shed in the feces of CPV-infected dogs.

Precautions and warnings

- All wastes should be properly decontaminated prior to disposal.
- No do not mix components from kits with different lot numbers.
- No do not use a SNAP device that has been activated prior to the addition of sample.
- The sampling swab is not designed to be used rectally.
- Lubricants may reduce test sensitivity by reducing the volume of feces tested.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.

Storage

- Store at 2-25°C until the expiration date.
- All components must be at room temperature (18-25°C) before running the test.
- Do not heat. This will take at least 30 minutes depending upon the temperature of your laboratory.

Kit components

| Item | Reagents | Quantity |
|---|---|----------|
| 1 | Swabs with anti-parvovirus: HRPO conjugate (Preserved with ProClin™ 150) | 5 |
| 2 | SNAP device | 5 |
| Reagents contained in each device: | | |
| Wash solution (preserved with ProClin™ 150) | 0.4 mL | |
| Substrate solution | 0.6 mL | |

Conjugate: H317/H412/P261/P280/P333+P313: May cause an allergic-skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear eye protection/protective clothing. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

WARNING
Was S1000: H317/H412/P261/P280/P333+P313: May cause an allergic-skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear eye protection/protective clothing. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention.

Sample information

- Samples must be at room temperature (18-25°C) before beginning the test procedure.
- Canine fecal matter is required for this test. Swabs are provided for sampling.
- Fecal samples can be stored at 2-8°C for 24 hours. If longer storage is required, samples should be frozen.

Test procedure

- If stored in a refrigerator, allow all components to equilibrate to room temperature (18-25°C) for 30 minutes. **Do not heat.**

2. Obtain a sampling swab and a SNAP device for each sample to be tested. Pull and twist the tube covering the swab tip to remove the tube from the swab/reagent bulb assembly (A). Using the swab, coat the swab tip with fecal material. Then, return the swab to the tube (B).

NOTE: Only a thin coat of fecal material on the swab is required; do not coat the swab with excess feces.


3. Break the purple valve stem inside the bulb assembly by bending the assembly at the narrow neck (C), rebranding the opposite way may be helpful. Squeeze the reagent bulb three times to pass the blue solution through the swab tip and mix it with the sample (D).

4. Place the SNAP device on a horizontal surface. Using the swab as a pipette, dispense 5 drops of the fluid into the sample well, being careful not to splash contents outside of the sample well.

The sample will flow across the result window, reaching the activation circle in 30-60 seconds. Some sample may remain in the sample well.

5. When color F1ST appears in the activation circle, push the activator firmly until it is flush with the device body.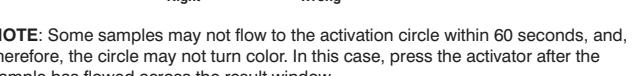

NOTE: Some samples may not flow to the activation circle within 60 seconds, therefore, the circle may not turn color. In this case, press the activator after the sample has flowed across the result window.

6. Read the test result at 8 minutes.

Interpreting test results

To determine the test result, read the reaction spots in the result window and compare the color intensity of the sample spot to that of the negative control spot.

Positive results
Color development in the sample spots that is darker than the negative control indicates a positive result, and the presence of parvovirus antigen in the sample.

Negative results
Color development only in the positive control spot indicates a negative result.

Invalid results
Negative control (safeguard against false-positives): If color in the negative control spot is equal to or darker than the color in the sample spot, the result is invalid and the sample should be retested.

No color development: If the positive control does not develop color, repeat the test.

Background: If the sample is allowed to flow past the activation circle, background color may result. Some background color is normal. However, if colored background obscures the test result, repeat the test.

CPV shed window:—The shed window for CPV is typically highest day 4 to day 7 post-infection and usually correlates with the onset of clinical signs. In puppies with moderate to severe clinical signs, the shed window may be delayed by 1-2 days relative to the onset of clinical signs. Veterinarians tend to waited by day 8-10 (post infection). It is important, therefore, to collect feces for viral detection at the onset of clinical illness and if negative for CPV, retest in 1-2 days.

Parvo vaccine cross-reactivity:—In a study of 64 dogs vaccinated with six different modified live CPV-2 vaccines, the SNAP Parvovirus Antigen Test Kit did not detect CPV-2 in their feces.

A population of 64 beagles with or no antibody to canine parvovirus-2 (CPV-2) were vaccinated with one of five different combination vaccines (Duramune® Max 5, Ft. Dodge Animal Health; Progard® 5, Intervet; Vanguard® Plus SL, Pfizer Animal Health; Recombitek® C4, Merial; Galaxy® DA 2PPv, Schering-Plough Animal Health) or one monovalent product (NeoPar® NEOTECH LLC) containing modified live CPV-2 vaccine. Fecal samples were collected on day 0 and on one or more of the following days 3, 4, 5, 6 and 7 post-vaccination. All samples were tested for CPV-2 using the SNAP Parvovirus Antigen Test Kit. No cross-reactivity was detected.

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Symbol descriptions

| | | |
|------------------------|---|---|
| Use by date |  ECREP | Authorized representative in the European Community |
| Batch code (lot) | | |
| Serial number |  I | Consult instructions for use |
| Temperature limitation |  IVD | In vitro diagnostic |
| Manufacturer |  dd | Date of manufacture |
| REF | Catalog number | |

IDEXX Technical Support

USA/Canada: 1 800 248 2453 • idexx.com

Australia: 1 800 44 33 99 • idexx.com.au

Europe: idexx.eu

References

D'Cunha, N. et al. "Maternally-derived antibodies in pups and protection from canine parvovirus infection." *Biology*. 2005; (33): 261-267.

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Patent information: idexx.com/patents

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© 2022 IDEXX Laboratories, Inc. Todos os direitos reservados.
*SNAP is a trademark of commerce or das marca depósita d'IDEXX Laboratories, Inc., o IDEXX é uma marca registrada da IDEXX Laboratories, Inc., ou das suas filiais em outros países.